

If you select the **Digital Image** option when creating your rule, the email notification (below, left) will contain a link to a report that itself contains a link to the images. The recipient clicks the link in the report (below, top right), and Techcyte Fusion AP opens, displaying the relevant images (below, bottom right).

The image is a composite of three parts illustrating a digital image workflow:

- Top Left (Email):** An email notification from Mayo Clinic Laboratories. The subject is "INT Test Results Available for 7 1 001*****017q". The body contains a link to a report: "001*****017q". A red box highlights this link, and a red arrow points from it to the report on the top right.
- Top Right (Report):** A laboratory report from Mayo Clinic Laboratories. The patient name is "Miss UNSOLICITED-ENVTWO, TECHCYTE R". The report title is "BOB-1 IHC, Tech Only". A red box highlights a link: "Supplemental data is available at: https://mcl-int-portal.mayo.edu/accounts/C7234577/orders/338b39f-0acc-4c23-978e-38814645598/tests/BOB1/images". A red arrow points from this link to the Techcyte interface on the bottom right.
- Bottom Right (Techcyte Fusion AP):** The Techcyte Fusion AP interface showing a digital image of a tissue section. The image is labeled "A-1-PR-DfDx uTV8i" and "0.5x". A scale bar indicates "2 mm". The interface also shows a slide tray with two slides: "A 1-PR-DfDx uTV8i" and "B 1-BOB1 uTAs".

For instructions on using Techcyte Fusion AP, see the [application user guide](#).